

HemaQuell®

RESORBABLE BONE HEMOSTAT

Description

HemaQuell® Hemostat is a resorbable, water soluble surgical implant material which is used for the control of bleeding from bone surfaces. It is a sterile blend of alkylene oxide copolymers.

Indications

HemaQuell® Hemostat is indicated as an implant for the control of bleeding from bone surfaces.

Mode of Action

HemaQuell® Hemostat achieves hemostasis by way of tamponade, physically blocking exposed bleeding bone surfaces with a coating of a malleable barrier that is resorbed in 2 – 7 days.

Contraindications

- Surgical sites that are infected.
- The use of the product to provide structural support to bone or to stop bone growth.

Warnings and Precautions

- HemaQuell® Hemostat is provided sterile for single use only: do not resterilize.
- Discard open, unused or damaged packages.
- Protect the product from heat. Store at 0 to 32 degrees C (32 to 90 degrees F).

Instructions for Use

1. HemaQuell® Hemostat is ready for use as delivered. It will soften slightly if manipulated by hand.
2. To extrude HemaQuell® Hemostat from the applicator tube, remove the cap, then rotate the wheel at the base of the tube clockwise to expose the desired amount of product. Do not rotate the wheel past the built-in stop or try to force the base out of the tube.
3. Apply HemaQuell® Hemostat directly to the bleeding bone surface using a crayon-like application. If necessary, reinforce anchorage by pressing the hemostat into the defect.
4. Alternatively, extrude HemaQuell® Hemostat from the tube, mold as needed, and apply by hand or with a surgical instrument.
5. Caution: since HemaQuell® Hemostat is water-soluble, aggressive irrigation at the application site may lead to breakthrough bleeding and reapplication may be required.
6. Depending on the size of the defect and the porosity of the bone, multiple tubes may be required.

Sterility

HemaQuell® Hemostat is sterilized via gamma irradiation. It is provided sterile in a clear envelop for delivery into a sterile field prior to use.

Learn more at HemaQuell.com
WNDM Medical, Inc.
1200 Summit Avenue, Suite 414
Fort Worth, TX 76102
800-205-7719
Manufactured in the USA

HemaQuell® is a trademark of WNDM Medical, Inc.
All rights reserved. Copyright © 2018 Rev. 180504



Reuse not allowed



Read usage instructions



Sterilized through radiation



Do not resterilize



MR safe



Protect from heat. Store at 0 degrees C to 32 degrees C

Product not made with natural rubber latex